# CLEAR CHOICE

## INSTRUCTION

### HCG Urine Pregnancy Test ONE • HCG Urine Pregnancy Test PLUS

• Model: Strip • For Self-Testing

**INTENDED USE:**  
HCG Urine Pregnancy Test Intended for Self-Testing, measures the presence of the hormone Human Chorionic Gonadotropin (hCG) in human urine for the early detection of pregnancy.

**SUMMARY AND EXPLANATION:**  
hCG is produced by the placenta during pregnancy, shortly after the embryo attaches to the uterine lining. Usually hCG can be detected by a home pregnancy test within 3-4 days after implantation.

So sensitive it may be used as early as 6 days before the day of your missed first period (5 days before day of expected first period), any time of day.  
The further into the pregnancy you are, the higher the levels of hCG in your urine. This pregnancy dip strip detects hCG in urine at a concentration of 10 mIU/ml (mIU=International Units) or greater. The concentration of hCG in non-pregnant women is normally 5.0 mIU/mL. At the time of the last missed menstrual period, urine hCG levels are about 100 mIU/ml with peak levels of 100,000 to 200,000 mIU/ml seen at the end of the first trimester.

**PACKAGE CONTENTS:**  
1 HCG Urine Pregnancy Test One: One foil pouch with 1 HCG Urine Pregnancy Strip Test, Desiccant (Discard - Do not eat), Instruction for use.

2 HCG Urine Pregnancy Test PLUS: Two foil pouches with 1 HCG Urine Pregnancy Strip Test each, Desiccant (Discard - Do not eat), Instruction for use.

**STORAGE AND STABILITY:**  
The test kit can be stored at temperature 2°C to 30 °C in the sealed pouch to the date of expiration. The test kits should be kept away from direct sunlight, moisture and heat.

- PRECAUTIONS:**
- 1. **FOR IN-VITRO DIAGNOSTIC (IVD) USE ONLY, NOT for internal use.**
  - 2. Read directions for use carefully before performing this test.
  - 3. Do not use beyond the expiration date.
  - 4. Do not reuse the test strip. Discard it in the dustbin after single use.
  - 5. Do not use the strip if the foil pouch is damaged.
  - 6. Hold the test strip by the handle only (coloured end).
  - 7. Once the foil pouch has been opened, the test strip should be used immediately. Prolonged exposure to ambient humidity will cause product deterioration.
  - 8. Treat urine samples and used test devices as if they are potentially infectious. Avoid contact with skin.
  - 9. Keep out of reach of children.


**TEST PROCEDURE:**  
**1. WHEN CAN I CARRY OUT THE TEST?**  
You can take this pregnancy test from 6 days before the day of your missed period (5 days before day of expected period), any time of day.

**2. SPECIMEN COLLECTION AND HANDLING**  
A urine specimen must be collected in a clean and dry container. The first urine specimen in the morning is preferred since it generally contains the highest concentration of hCG; however, urine specimens collected at any time of the day may be used.

- 3. TO CARRY OUT TEST**
- Bring the sealed pouch to room temperature. To begin testing, open the sealed pouch and remove the strip. Do not touch the arrow end and the middle part of the strip.
  - Place the test strip vertically (straight) into the urine sample; make sure the arrows are pointing down. Do not allow the urine surface to go above the MAX (maximum) level line (marked by arrows) on the test strip.
  - Remove the strip from the urine after about 10 seconds and lay the strip on a clean, dry surface.
  - Wait for coloured bands to appear. Depending on the concentration of hCG in the urine specimen, positive results may be observed in as short as 40 seconds. However, to confirm a negative result, the complete reaction time (5 minutes) is required. Do not read results after 10 minutes.
  - Discard the test after use. Put the used test into special container if possible and regard it as if is infectious. Don't litter.

**4. INTERPRETATION OF RESULTS**  
**NEGATIVE:** Only one visible coloured band in control region, no coloured band in test region, which indicate the result of the test is negative, no hCG has been detected in the urine.  
This means you are either not pregnant or you have tested too early. If you are not sure repeat the test in 48 hours.

**POSITIVE:** Two visible coloured bands in both test region and control region, which indicate the result of the test is positive, then this means there is a strong possibility that you are pregnant — hCG has been detected in your urine.

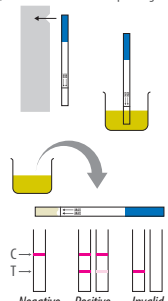One line may be lighter than the other, do to this still is a positive result.  
**INVALID:** No visible coloured band(s) at all, or only one coloured band in test region, which indicate the kit is invalid. Repeat the test with a new test kit!

**REAGENTS:**  
Test device comprised colloidal gold coated with anti-β-HCG antibody; NC membrane coated with mouse anti-α-HCG antibody and rabbit anti mouse IgG.

**LIMITATION OF THE PROCEDURE:**  
1. Alcohol may interfere with the test result. It is not recommended to use the test after drinking alcohol.  
2. Occasionally urine specimens containing less than 10 mIU/ml can also yield positive results.  
3. If you test too early, hCG level may still be low and so you give a negative result. In this case, another urine specimen should be obtained at least 48 hours later and tested.  
4. hCG levels may remain detectable for several weeks after normal delivery, delivery by caesarian section, spontaneous abortion or therapeutic abortion.  
5. In cases where very high levels of hCG are present (>500,000 mIU/ml) a false negative result can occur due to a "Prozone" effect. If pregnancy is still suspected, simply dilute specimen 1:1 with deionized water and retest.  
6. If an urine sample is too dilute (i.e., low specific gravity) it may not contain a representative level of hCG. If pregnancy is still suspected, a first morning urine sample should be obtained and retested 48 hours later.  
7. As is true with any diagnostic procedure, the user should evaluate data obtained by using this kit in light of other clinical information and consult their doctor for the final diagnosis of pregnancy before making any decision of medical relevance.

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